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en

Intended purpose

The product is used for the transport, processing (cleaning, disinfection, sterilization), storage and provision of medical instruments. The product is intended for reuse and processing by the user.

ATTENTION

The product must be cleaned before first use.

The product can be heated during processing. There is a risk of burns on contact with the hot product.

- ▶ Wear heat-insulating protective gloves.
- ▶ Allow the product to cool down sufficiently.

WARNING

Risk of injury and/or malfunction!

- ▶ Check the product for loose, bent, broken, cracked, worn or broken parts before each use.
- ▶ Carry out a function test before each use.

WARNING

Risk of trapping or damage to gloves when handling the instruments!

- ▶ Handle instruments with care.

WARNING

Risk of infection from contaminated products!

- ▶ Comply with national regulations when disposing of or recycling the product, its components and their packaging.

Processing instructions

Before first use, the product must be reprocessed with the steps of cleaning, disinfection and sterilization.

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General information

Contact with the following substances reduces the service life of the product and can jeopardize effective reprocessing:

- Substances containing salt and sulfur
- Chlorides (salt)
- Extraneous rust, e.g. contamination from reprocessing

Follow the cleaning agent manufacturer's instructions.

Report any serious incidents that occur in connection with the product to the manufacturer and the responsible authority.

Performance features

The materials used are intended for processing (cleaning, disinfection, sterilization) and can withstand temperatures of up to 150°C.

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