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en

Intended use

The MedProcessTec Veress pro.Unit is used to clean and disinfect reusable stainless steel Veress needles.

ATTENTION

The product must be cleaned before first use.

WARNING

Risk of injury and/or malfunction!

- ▶ Check the product for loose, bent, broken, cracked, worn or broken parts before each use.
- ▶ Carry out a function test before each use.

WARNING

Risk of trapping or damage to gloves when handling the instruments!

- ▶ Handle instruments with care.

WARNING

Risk of infection from contaminated products!

- ▶ Comply with national regulations when disposing of or recycling the product, its components and their packaging.

Processing instructions

Before first use, the product must be reprocessed with the steps of cleaning, disinfection and sterilization.

General information

Contact with the following substances reduces the service life of the product and can jeopardize effective reprocessing:

- Substances containing salt and sulfur
- Chlorides (salt)
- Extraneous rust, e.g. contamination from reprocessing

Follow the cleaning agent manufacturer's instructions.

Report any serious incidents that occur in connection with the product to the manufacturer and the responsible authority.

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Application

The MedProcessTec Veress **pro**.Unit is intended for Veress needles to be cleaned and disinfected in the automatic process of a washer-disinfector.

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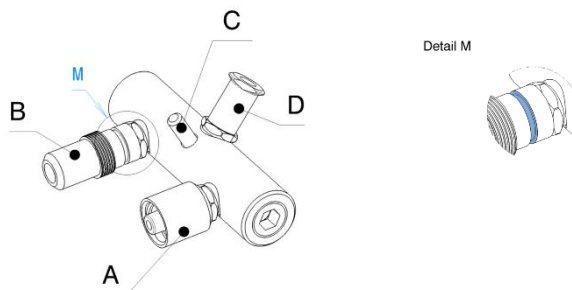
Fig. 1: Components of the Veress **pro**.Unit

- A** - Connection for stylet with Luer-Lock compatible attachment
- B** - Connection for hollow needle with fine thread
- C** - Fixing pin for two-way stopcock with 3 mm inner diameter
- D** - Connection for silicone tubing with Luer-Lock compatible attachment and connection to washer-disinfector (WD)
- M** - Marking for the hollow needle for pre-cleaning

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Disassemble the Veress needle and remove coarse soils with tap water.

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Connect the individual parts of the Veress needle to the Veress **pro**.Unit as follows (Fig. 1 + Fig. 2).

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- Connect the stylet with Luer-Lock compatible attachment to connection **A**.
- Connect the hollow needle with fine thread to connection **B** and screw up to mark **M**.
- Attach the two-way stopcock (3 mm inner diameter) to pin **C** and screw the clamping screw onto pin **C**.
- Using a water pressure gun, flush the components of the Veress needle through connection **D** for 3 x 5 seconds while holding the openings for the two-way valve on the stylet closed with two fingers. Then screw the hollow needle with fine thread onto connection **B** as far as it will go.
- Then place the Veress **pro**.Unit with the components of the Veress needle in a wire basket and connect port **D** to the WD using a silicone tube with a Luer-lock-compatible attachment.

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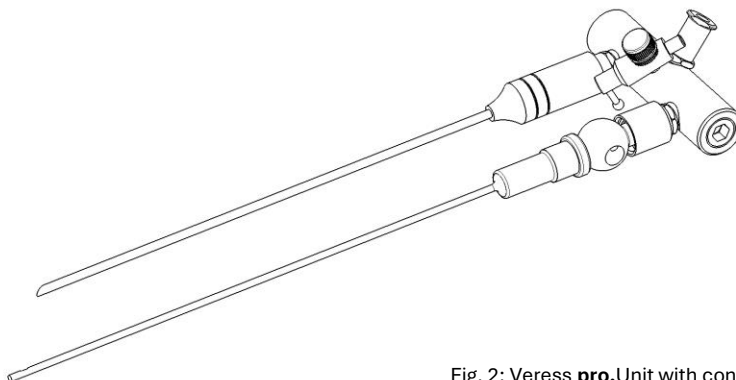
Fig. 2: Veress **pro**.Unit with connected Veress Needle